



## ihcDirect® CK19 Ab Anti-Human CK19 (Clone R1002)

Ab: K32033-015, 150 tissue stains\*

Ab: K32033-005, 50 tissue stains\*

### Intended Use: For *In Vitro* Diagnostic Use

Polymerized horseradish peroxidase (polyHRP)-labeled anti-CK19 (R1002) rabbit monoclonal antibody is intended for laboratory use to qualitatively identify by light microscopy the presence of CK19 in sections of formalin-fixed, paraffin-embedded (FFPE) and/or cryostat tissues using immunohistochemistry (IHC) test methods. The clinical interpretation of any staining or its absence should be complemented by morphological studies using proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests and proper controls interpreted by a qualified pathologist and/or physician. This conjugate has been pre-diluted and optimized for IHC use without further dilution.

### Summary and Explanation:

ihcDirect® CK19 is a polyHRP conjugated rabbit monoclonal antibody (clone R1002). CK19 is the smallest cytokeratin with a molecular weight of 40 kDa. It is often coexpressed with CK7 and present in both simple and complex epithelium. Cytoplasmic and membrane positive staining for CK19 is seen in normal tissues, such as, hair follicles, sweat gland, mammary glands, esophagus, bile ducts, GI tract, respiratory and squamous epithelium as well as renal tubules, exocrine pancreas and trophoblasts. In the malignant tissues, the CK19 positive expression is seen in breast cancer, endometrial cancer, hepatoid adenocarcinoma, renal carcinoma, urothelial carcinoma, lung cancer, Paget's disease, pancreatic ductal carcinoma, squamous cell carcinoma and thyroid papillary carcinoma. The antibody may also be useful as part of thyroid cancer panel. CK19 has been used to confirm diagnosis of papillary thyroid carcinoma in cytology or equivocal cases and helps distinguish the follicular variant of papillary thyroid carcinoma from follicular adenoma, hyalinizing trabecular adenoma, Grave's disease, and some multinodular goiter. The antibody may also be used to distinguish hepatocellular carcinoma from either hepatoid adenocarcinoma metastatic to liver or cholangiocarcinoma.

### Principle of Procedure:

The ready-to-use ihcDirect polyHRP CK19 antibody conjugate is directly applied to pretreated tissue sections, where it binds to CK19. A Working Solution (WS) of a chromogen such as ihc DAB 1:1 is then applied to the tissue. The antibody-linked pHRP reacts with the chromogen to form a visible colored product at the site of the human CK19 location. The specimen may then be counterstained and a coverslip applied. Results are viewed and interpreted using a light microscope. Volumes are based upon 100µl antibody per tissue. This product may be used to perform IHC manually or on an open automated IHC staining system.

### Reagents Provided:

Part No.	∇	Description
K32033-015	150*	15ml of ihcDirect CK19 Ab ready-to-use antibody conjugate. See Ancillary Reagents section.
K32033-005	50*	5ml of ihcDirect CK19 Ab ready-to-use antibody conjugate. See Ancillary Reagents section.

\* At estimated volume of 100µl of antibody conjugate per tissue

Immunogen	Clone	Species	Ig Class	Total Protein Conc.
Human CK19	R1002	Rabbit	IgG	10 mg/ml

CK19 antibody is a rabbit monoclonal antibody to human CK19 purified from animal origin-free cell culture supernatant. HRP is extracted from horseradish plant.

Novodix ihc Blocker and chromogens such as ihc DAB 1:1 Kit, ihc Magenta 1:1 Kit, and DAB Kit are recommended for use with the CK19 antibody.

### CK19 pHRP Antibody (K32033-###):

Reagent Description	Component Part Numbers	Sizes (ml)
CK19 pHRP	H32033-(R###) (005, 015)	5, 15

### Ancillary Reagents for Use with CK19 Antibody:

Reagent Description	Part Numbers	Sizes (ml)
ihc Blocker Intl. USA	K51001-(015) (OR)	15
	K51002-(015, 030)	15, 30
ihc DAB 1:1 Kit	K50002-### (015, 030)	15, 30
ihc Magenta 1:1 Kit USA	K50011-### (015, 030)	15, 30
DAB Kit	K50001-### (015, 030)	15, 30

### Materials Needed but Not Provided:

The following reagents/supplies may be required in staining but are not provided:

- |   |  |
|---|--|
| 1. Frozen section fixative, Acetone or NBF (10% NBF§) | 9. Peroxide blocker (optional)   |
| 2. Positive and negative control tissues              | 10. Instruments used for tissue pretreatment, such as water bath, or pressure cooker or microwave oven (when using FFPE tissues) |
| 3. Microscope slides, positively charged (required)   | 11. Hematoxylin  |
| 4. Staining jars, baths or processing tools           | 12. Xylene or Xylene substitute  |
| 5. ihc Wash Buffer (PBS-T)                            | 13. Ethanol  |
| 6. Pipettor and pipet tips                            | 14. Mounting medium  |
| 7. Timer  | 15. Cover slips  |
| 8. Antigen retrieval buffer (when using FFPE tissues) | 16. Light microscope (40 - 400x)   |

§ NBF – neutral buffered formalin

### Novodix Bulk Reagent Formulations:

- ihc Wash Buffer (PBS-T), (10 mM phosphate buffer, pH 7.2, 150 mM NaCl, 0.05% Tween-20).
- ihc Antigen Retrieval Buffer (10 mM Citric buffer, pH 6.0, 0.02% Tween 20).

### Storage and Handling:

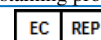
This product should be stored at 2-8°C. and is suitable for use until expiry date when stored at that temperature. Do not freeze. Do not use the product after expiration date unless dating extension information is provided by Novodix. If the reagent is stored under any conditions other than those specified in the package insert, they must be verified by the user.

### Specimen Preparation:

**Paraffin Sections:** Tissues routinely processed with 10% NBF are suitable for use prior to paraffin embedding. Consult references (Kiernan, 1981; Sheehan & Hrapchak, 1980). Variable results may occur as a result of prolonged fixation. Each section should be cut to the appropriate thickness (approximately 4-5µm) and placed on a positively charged glass slide. Slides containing the tissue section may be baked for at least one hour but not exceeding 24 hours in a 58-60°C±5°C oven. Osseous tissues should be decalcified prior to tissue processing to facilitate tissue cutting and prevent damage to microtome blades (Kiernan, 1981; Sheehan & Hrapchak, 1980).

**Frozen Tissue Sections:** Frozen tissue is sectioned to the appropriate thickness (approximately 5 µm) and placed on a positively charged glass slide. Tissues should be fixed in either reagent grade acetone or 10% NBF for 1-2 minutes immediately after sectioning. A short time to air dry may help tissues adhere to the slide. Other fixatives should be validated prior to use. Following fixation, tissues should be processed within a few minutes or tissues may be stored in PBS for as long as a day.

**Treatment of Tissues Prior to Staining:** Pretreatment is tissue dependent and should be performed as suggested in the staining procedure sections.



### Warnings and Precautions:

1. Read and understand all of the Novodiox Instructions for Use (IFUs) before product use.
2. The CK19 pHRP antibody conjugate is pre-diluted. Further dilution may reduce signal intensity or increase false-negative staining. These recommendations are for guidance only. Laboratory managers should determine their own procedures and quality policies.
3. To obtain best results when working with frozen tissues, it is desirable to freeze tissues as quickly as possible following extraction.
4. Use caution and shorten incubation times when utilizing intense hematoxylin counterstains such as Gills as these stains may tend to mask antibody staining.
5. Take reasonable precautions when handling reagents. Use protective equipment such as disposable gloves and lab coats when handling materials. Read Safety Data Sheets (SDS) prior to use.
6. Avoid contact of reagents with eyes and mucous membranes. If reagents come in contact with sensitive areas, wash with copious amounts of water.
7. Use charged slides to secure tissues appropriately.
8. Patient specimens and all materials that come into contact with patient specimens should be handled as bio-hazardous materials and disposed of appropriately.
9. Consult local or state authorities with regards to recommended methods of disposal of bio-hazardous and hazardous chemical waste materials.
10. Incubation time and temperature other than those specified may give erroneous results. The user must validate any such changes.
11. Use lab grade quality chemicals such as NBF and water when preparing reagents. Users should validate performance including stability for laboratory prepared reagents (at 1X).
12. Avoid microbial contamination of reagents.
13. Fixation is a vital part of the protocol and fixation times may vary with the fixative chosen, tissue type, e.g. containing fat and other parameters. Generally, an acetone or NBF fixation of 1-2 minutes is recommended. Place frozen tissue sections into fixation solution shortly after sectioning.
14. **Prolonged exposure to room or freezing temperatures may alter targeted epitopes.** It is best to prevent slides from drying out during the staining process to avoid unwanted background staining.

### Staining Procedures:

#### General Operating Notes:

1. Equilibrate all reagents to room temperature prior to use. Swirl or shake the ihc Blocker and pHRP-labeled antibody solutions before use. **Do not vortex.** Calculate the amount of chromogen WS needed (100µl per tissue) and **freshly** prepare chromogen WS. See instructions for use.
2. Gently and thoroughly wash tissues during manual wash steps. Avoid direct high velocity streams of wash that might tend to damage or cut delicate tissues.
3. Following each manual assay step, remove excess fluids on tissue slides with tissue paper. Excessive residual solution may dilute subsequent reagents, causing negative or uneven staining.
4. To reduce background signal, wash thoroughly following antibody step.
5. For the tissues with high oxidase activity, e.g. gastrointestinal or renal tissues, an additional blocking step with H<sub>2</sub>O<sub>2</sub> is required to minimize background.
6. The following protocol has been validated at temperatures between 21°-30°C (70°-86°F) for incubating ihc Blocker, CK19 Ab pHRP and chromogen WS. If room temperature is less than 21°C, users may incubate labeled antibody for a longer period of time (≤5 minutes depending upon temperature). Consistent results have been obtained using a slide warmer set to 30°C at the surface of the slide.

#### Frozen Tissue Sections:

1. Following fixation, rinse slides with 1X ihc Wash buffer and then wipe away any excess fluid with a Kimwipe® or paper towel.
2. Dispense 100µl of ihc Blocker, covering the entire tissue, and incubate at room temperature for 1 minute. Then firmly tap off any excess ihc Blocker onto an absorbent surface but do not rinse the slides.
3. Dispense 100µl of pHRP antibody, covering the entire tissue, and incubate for 3 minutes at room temperature. To obtain a darker stain, users may extend incubation up to a total of 5 minutes. Then, rinse slides with 1X ihc Wash buffer and wipe away any excess fluid.
4. Dispense 100µl of a chromogen WS, such as DAB covering the entire tissue, and incubate for 1-3 minutes at room temperature. Users should determine the optimal incubation time for their particular chromogen or lab environment.

Then rinse slides with either 1X ihc Wash buffer or lab grade water and wipe away any excess fluid.

5. Add a counterstain. Incubation times will vary according to the counterstain formulation. Then, rinse slides with water and wipe away any excess fluid.
6. Apply aqueous media or dehydrate slides with the user's typical dehydration protocol and then add coverslip.

#### Test Timing Est. (10-minute IHC protocol for frozen tissue sections):

ihcDirect Frozen Tissue Procedure	Time in minutes
Fix with Acetone or Neutral Buffered Formalin	Start
- Wash with <input type="text" value="ihc Wash"/> remove excess fluid	
Block with <input type="text" value="ihc Blocker"/>	1
- Tap+Absorb to remove excess blocker	- - -
Novodiox ihcDirect pHRP Antibody	3
- Wash thoroughly with <input type="text" value="ihc Wash"/>	
- Tap+Absorb to remove excess wash buffer	- - -
Novodiox Chromogen Working Solution	1-3
- Wash with <input type="text" value="ihc Wash"/> or DI water	
Hematoxylin counterstain (Conc. Dependent)	2-45 sec
Wash with water	
Dehydrate/Mounting Medium and Coverslip	User Det.
<b>Total</b>	<b>~10</b>

#### Paraffin Tissues:

1. Deparaffinization: Soak slides in Xylene 3 times for 5 minutes each. Next, 3 minutes each in 100%, 95% and 75% ethanol. Then wash slides with tap water in slide tank for two times, 2 minutes each time.
2. Antigen retrieval: Using a water bath, incubate slides in antigen retrieval buffer in a slide tank at 96°C for 30 minutes, then cool the slides down to room temperature for 30 minutes. Rinse the slides twice with tap water, 2 minutes each time.
3. (Optional) Block tissue with H<sub>2</sub>O<sub>2</sub>: Soak the slides in 3% H<sub>2</sub>O<sub>2</sub> in slide tank, stand for 10 minutes. Rinse the slides with tap water twice and then wash once with PBS-T in slide tank for 2 minutes.
4. Dispense 100µl of ihc Blocker covering the entire tissue and incubate at room temperature for 15 minutes. Remove ihc Blocker as much as possible but do not rinse the slides with PBS-T or water.
5. Dispense 100µl of pHRP labeled anti-human CK19 antibody on slides covering the entire tissue and incubate for 15-30 minutes at room temperature. Rinse the slides three times with PBS-T in slide tank, 2 minutes each time. Note: Place slides in a wet chamber to prevent evaporation if longer incubation times are used.
6. Dispense 100µl of chromogen WS covering the entire tissue, incubate for 3-10 minutes at room temperature. Rinse the slides twice with tap water in slide tank, 2 minutes each time.
7. Counterstaining: Add hematoxylin and incubate for 1 minute at room temperature. Rinse twice with tap water for 2 minutes, each time.
8. Dehydration: Soak slides in the following order: 75% ethanol for 3 minutes, 95% ethanol for 3 minutes, 100% ethanol for 3 minutes and Xylene twice at 5 minutes each time.
9. Applying Coverslip: Add one drop of permanent mounting medium on both the slide and the coverslip, then apply coverslip.

#### Quality Control Procedures:

Positive and negative controls should be run simultaneously with patient specimens.

**Positive Tissue Control:** The recommended positive control tissues for this antibody are known CK19 Ab positive tissues. One positive tissue control for each set of test conditions should be included in each staining run. Previous tissue



specimens that have been frozen and freshly cut or in some cases, an individual's own tissue may be used as a control.

The tissues used for the positive control should be selected from patient specimens with well-characterized low levels of the positive target activity that gives weak positive staining. Known positive tissue controls such as breast, lung and colon should only be utilized for monitoring the correct performance of processed tissues and test reagents, rather than as an aid in formulating a specific diagnosis of patient samples. If the positive tissue controls fail to demonstrate positive staining, results with the patient specimens should be considered invalid.

**Negative Tissue Control:** The same tissue used for the positive control may be used as the negative tissue control. The variety of cell types in most tissue sections offers internal negative control sites. But this should be verified by the user. The components that do not stain should demonstrate the absence of specific staining and provide an indication of non-specific background staining. If specific staining (false positive staining) occurs in the negative tissue control, results with the patient specimens should be considered invalid.

**Troubleshooting:**

If an unexpected staining pattern occurs on control tissues or patient samples, consider the following:

- No staining:** If no staining is evident on positive control slide, please verify whether (1) chromogen WS was prepared freshly and correctly, (2) reagents were applied in the correct order, (3) pHRP-labeled antibody was indeed added, and (4) for FFPE tissue, dewaxing and antigen retrieval were performed adequately. Perform any corrective actions required and then repeat the procedure.
- Low signal or faint staining:** Please check whether (1) the reagents are not expired, (2) temperature of the testing environment was at least 21°C or a 30°C slide warmer was used, (3) chromogen WS was prepared freshly and correctly, (4) excess ihc Wash solution was not left on the slide, causing subsequent reagents to be diluted, and (5) for FFPE tissue, dewaxing and antigen retrieval were performed adequately, and (6) for frozen tissue, acetone was used as a fixative (Acetone fixation may generate weaker signals). Perform any required corrective actions and repeat the procedure. Alternatively, if using a DAB chromogen, consider using another stain, e.g. ihc Magenta 1:1 to obtain more vibrant staining. In addition, some individuals may naturally have low expression of certain antigens. In these cases, users may extend the antibody incubation times by 1-2 minutes.
- High background:** Possible causes include (1) insufficient washing, (2) ihc Blocker not being applied or washed out after application, (3) specimens drying out, (4) prolonged chromogen incubation, (5) prolonged pHRP-labeled antibody incubation, and (6) specimens containing high level of endogenous peroxidase, which necessitates an additional blocking step (refer to the Staining Procedures for Paraffin Tissues). Perform any required corrective actions and repeat the procedure.

If an unexpected staining pattern is observed on control tissues or patient samples which cannot be explained by variations in laboratory procedures or a problem with the antibody is suspected, contact NovodiAx Technical Support or your local distributor immediately. Within the US and Canada call 1 (888) 439-2716 ext. 2 or 1 (510) 342-3043 ext. 2.

**Expected Results:**

Intense color stains the tissue with a clean background if CK19-expression cells exist. There will be no color staining if no CK19-expression cells exist in the tissue. Interpretation of the staining result is solely the responsibility of the user.

**General Limitations:**


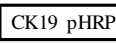
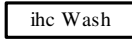


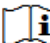
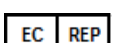


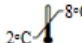

Immunohistochemistry is a multistep diagnostic process that requires specialized training in the selection of the appropriate reagents; tissue selection, fixation, and processing; preparation of the IHC slide; and interpretation of the staining results. Improper fixation, freezing, thawing, washing, drying, heating, sectioning or contamination with other tissues or fluids may produce artifacts, antibody trapping, or false negative results. Inconsistent results may be due to variations in fixation and embedding methods, or to inherent irregularities within the tissue (Nadji M, Morales AR. 1983).

The manufacturer provides these antibodies/reagents at optimal dilution for use following the provided instructions for IHC on prepared tissue sections. Any deviation from recommended test procedures may invalidate declared expected results; appropriate controls must be employed and documented. Users who deviate

from recommended test procedures must accept responsibility for interpretation of patient results under these circumstances.

**Performance Characteristics:**

The ihcDirect CK19 pHRP test performance has been determined using both frozen and FFPE tissue sections. NovodiAx has conducted studies to evaluate the performance of the antibody conjugates and recommended ancillary reagents and supplies. The antibodies and systems have been found to be sensitive and show specific binding to the antigen of interest with minimal to no binding of non-specific tissues or cells. NovodiAx antibodies and ancillary reagents have shown reproducible and consistent results when used within a single run, between runs and between lots. These products have been determined to be stable for the periods of time specified on the labels either by standard real-time and/or accelerated methods. NovodiAx ensures product quality by testing each lot of material and by testing materials at regular intervals and via surveillance programs.

ihcDirect CK19 Key to Symbols			
	<i>In vitro</i> diagnostic medical device		pHRP Cytokeratin 19 antibody conjugate
	Catalog Number		Wash Buffer - 1X preparation
	Use by: YYYY-MM-DD		Manufacturer
	Consult Instruction for Use		Authorized European Representative
	Batch Code		CE Mark
	Temperature Limitation		Contains sufficient for <n> tests

**Instructions for Use (IFU) Access:**

To obtain the latest electronic version of an IFU document, visit our website at <https://www.novodiAx.com/literature/instructions-for-use-ifu/>. Printed copies of an IFU document may be obtained by contacting NovodiAx Technical Support or your local distributor.

**Bibliography:**

- Kiernan JA. *Histological and Histochemical Methods: Theory and Practice*. New York: Pergamon Press 1981.
- Sheehan DC and Hrapchak BB. *Theory and Practice of Histotechnology*. St. Louis: C.V. Mosby Co. 1980.
- Nadji M, Morales AR. Immunoperoxidase, part I: the techniques and its pitfalls. *Lab Med*. 1983; 14:767-771.
- Jerome Marson V et al. Expression of TTF-1 and cytokeratins in primary and secondary epithelial lung tumours: correlation with histological type and grade. *Histopathology*. 2004; 45(2): 125-34.
- Stroescu C., et al. The diagnostic value of cytokeratins and carcinoembryonic antigen immunostaining in differentiating hepatocellular carcinomas from intrahepatic cholangiocarcinomas. *J Gastrointest Liver Dis.*, 2006;(1): 9-14.
- Durnez A. et al. The clinicopathological and prognostic relevance of cytokeratin 7 and 19 expression in hepatocellular carcinoma. A possible progenitor cell origin. *Histopathology*. 2006; 49(2)L 138-5
- Maki M et al. Expression of cytokeratin 1, 5, 14, 19 and transforming growth factors-beta1, beta2, beta3 in osteofibrous dysplasia and adamantinoma: A possible association of transforming growth factor-beta with basal cell phenotype promotion. *Pathol Int*. 2000; 50(10): 801-7.

